

SETUP ICEPT TRAINING

Download to iPad

1. Requires **iPad2** or higher.
2. Works on iPad Mini (7.9") but the standard 9.7" screen is recommended.
3. Requires **iOS 7.0+** or higher for the app to be stable.

For help on how to update your operating system click on the link below:

<http://www.wikihow.com/Update-iOS-Software-on-an-iPad>

4. To download **iCept Training** go to the App Store and search under "**icept testing**".
5. Select the **ICEPT TRAINING** button.




***** MULTI-LICENCE USERS SKIP TO STEP 12 *****

6. Select the **LOGIN** button in the top right corner and select “**Create Account**”. After creating an account an email will be sent to you to **confirm** your id. *If you do not receive an email try checking your Junk Email box.* For help, contact VFL customer support.




7. Enter email and password.

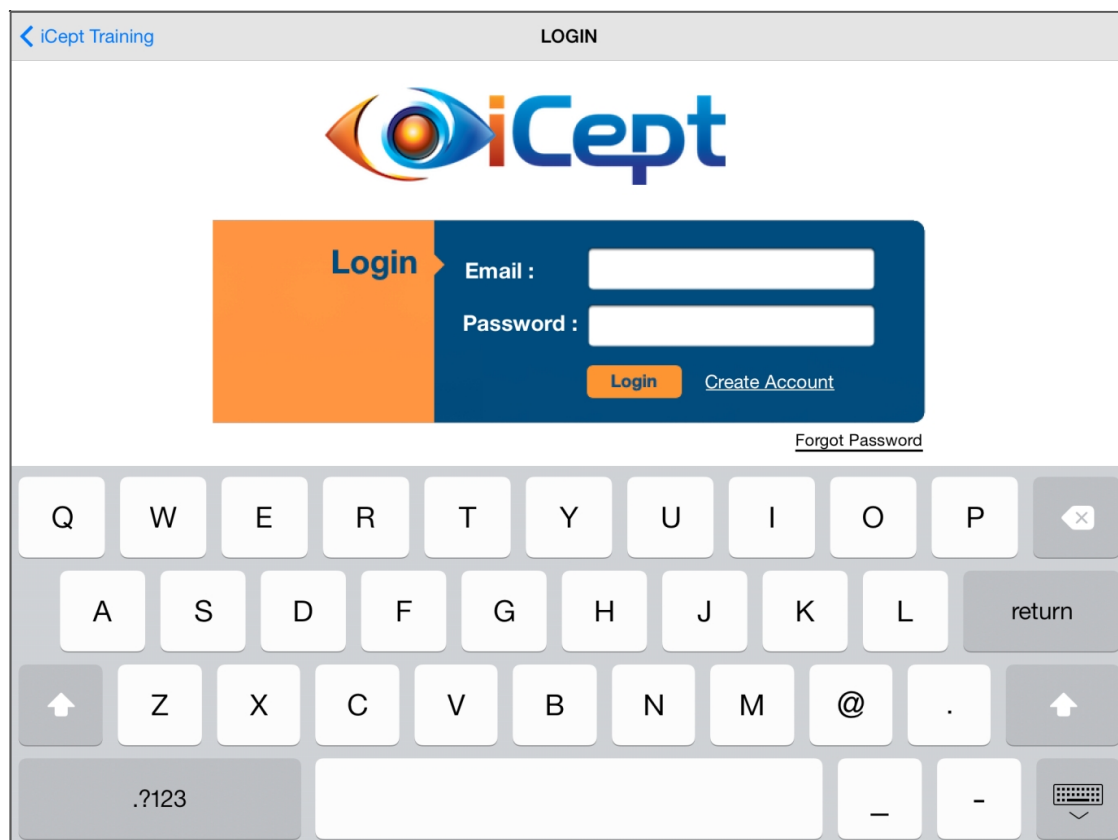

8. Select “Start”.



9. View the iCept Training dashboard of tasks for training and select the “Purchase” button.

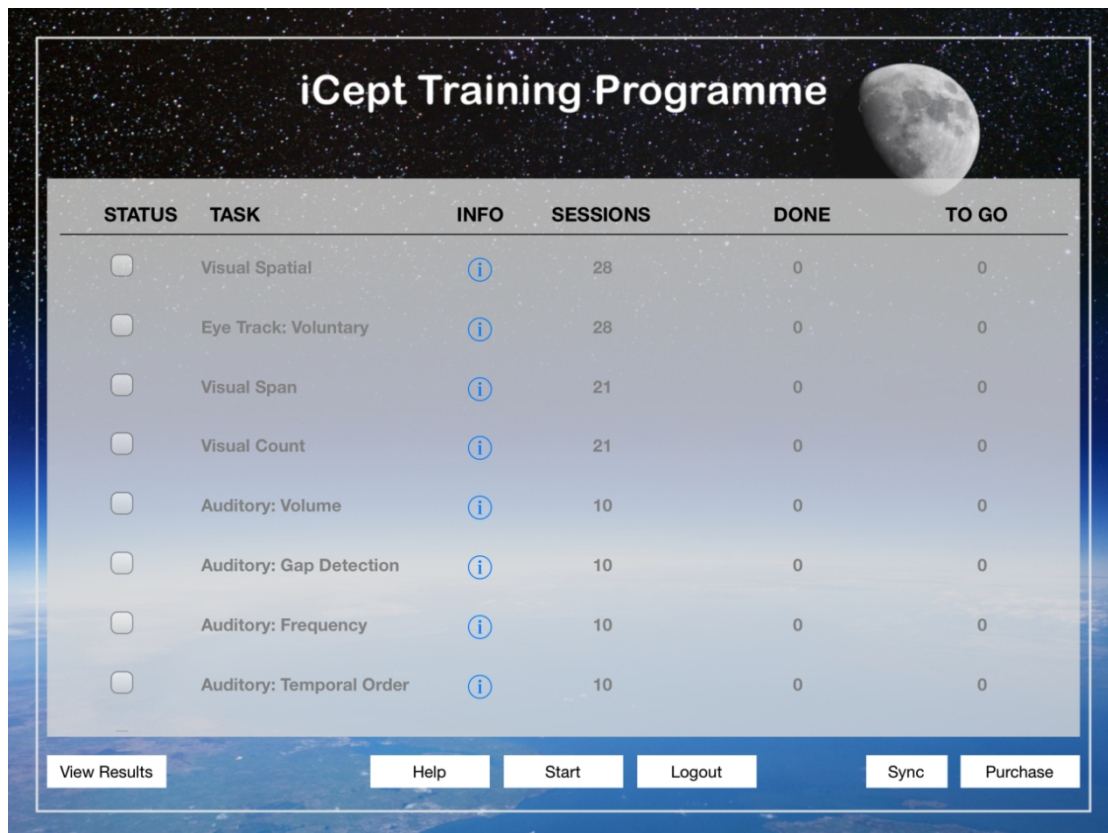The image shows the iCept Training Programme dashboard. The title 'iCept Training Programme' is at the top right, next to a moon graphic. Below the title is a table with columns: STATUS, TASK, INFO, SESSIONS, DONE, and TO GO. The table lists eight tasks, each with a checkbox in the STATUS column, an information icon in the INFO column, and numerical values in the SESSIONS, DONE, and TO GO columns. At the bottom of the dashboard are several buttons: View Results, Help, Start, Logout, Sync, and Purchase.

STATUS	TASK	INFO	SESSIONS	DONE	TO GO
<input type="checkbox"/>	Visual Spatial	i	28	0	0
<input type="checkbox"/>	Eye Track: Voluntary	i	28	0	0
<input type="checkbox"/>	Visual Span	i	21	0	0
<input type="checkbox"/>	Visual Count	i	21	0	0
<input type="checkbox"/>	Auditory: Volume	i	10	0	0
<input type="checkbox"/>	Auditory: Gap Detection	i	10	0	0
<input type="checkbox"/>	Auditory: Frequency	i	10	0	0
<input type="checkbox"/>	Auditory: Temporal Order	i	10	0	0

View Results Help Start Logout Sync Purchase

10. Check the tasks for training are the same as in your **iCept Report**.

Note: you can unselect a task before making a purchase or select new tasks later.

[Back](#)

PURCHASE

Confirm and purchase training for Stuart Warren

Reset Password

PURCHASE	TASK	SESSIONS (Days)	ADD SESSIONS	CLICK TO ADD
<input type="checkbox"/>	Visual Spatial	28	<input type="text"/>	Add Sessions
<input checked="" type="checkbox"/>	Eye Track: Voluntary	28	<input type="text"/>	Add Sessions
<input type="checkbox"/>	Visual Span	21	<input type="text"/>	Add Sessions
<input type="checkbox"/>	Visual Count	21	<input type="text"/>	Add Sessions
<input type="checkbox"/>	Auditory: Volume	10	<input type="text"/>	Add Sessions
<input type="checkbox"/>	Auditory: Gap Detection	10	<input type="text"/>	Add Sessions
<input checked="" type="checkbox"/>	Auditory: Frequency	10	<input type="text"/>	Add Sessions
<input type="checkbox"/>	Auditory: Temporal Order	10	<input type="text"/>	Add Sessions
<input type="checkbox"/>	Eye Track: Fixation	0	<input type="text"/>	Add Sessions
<input type="checkbox"/>	Eye Track: Reflex	0	<input type="text"/>	Add Sessions

Total Cost: 100.00 NZD

Change Currency:
NZD

Purchase

11. Pay for training using **PayPal**. You do not have to be a member!

- Login to PayPal account or
- Pay by credit card.

[Back](#)

PURCHASE

Confirm and purchas

Reset Password

PURCHASE	TASK	CLICK TO ADD
<input type="checkbox"/>	Visual Sp	Add Sessions
<input checked="" type="checkbox"/>	Eye Track	Add Sessions
<input type="checkbox"/>	Visual Sp	Add Sessions
<input type="checkbox"/>	Visual Co	Add Sessions
<input type="checkbox"/>	Auditory:	Add Sessions
<input type="checkbox"/>	Auditory:	Add Sessions
<input checked="" type="checkbox"/>	Auditory:	Add Sessions
<input type="checkbox"/>	Auditory:	Add Sessions
<input type="checkbox"/>	Eye Track	Add Sessions
<input type="checkbox"/>	Eye Track	Add Sessions

Total Cost: 100.00 N

Change Currency:
NZD

Purchase

Cancel

Payment

Payment for ICEPT TRAINING subscriptions \$100.00

PayPal Log In

12. MULTI-LICENCE USERS ONLY

Go to **LOGIN** in the top right hand corner and enter your email and password in the Login box. Select **“Login”** followed by **“Start”** to begin iCept Training.

13. The tasks purchased are shown in RED and the status box checked. The task currently being trained is shown in GREEN.



The screenshot shows the iCept Training Programme interface. It features a table with columns: STATUS, TASK, INFO, SESSIONS, DONE, and TO GO. The background is a space-themed image with a moon. At the bottom, there are buttons for View Results, Help, Start, Logout, Sync, and Purchase.

STATUS	TASK	INFO	SESSIONS	DONE	TO GO
<input type="checkbox"/>	Visual Spatial	i	28	0	0
<input checked="" type="checkbox"/>	Eye Track: Voluntary	i	28	0	28
<input type="checkbox"/>	Visual Span	i	21	0	0
<input type="checkbox"/>	Visual Count	i	21	0	0
<input type="checkbox"/>	Auditory: Volume	i	10	0	0
<input type="checkbox"/>	Auditory: Gap Detection	i	10	0	0
<input checked="" type="checkbox"/>	Auditory: Frequency	i	10	0	10
<input type="checkbox"/>	Auditory: Temporal Order	i	10	0	0

View Results Help Start Logout Sync Purchase

IMPORTANT: Training should start at the top and progress down. Each task should be completed BEFORE starting the next task!